

The Complete Fetal Scalp Blood Sampling Kit

Including the LiNA™ Fetal Blood Sampling Wand

The Complete Fetal Scalp Blood Sampling Kit contains everything necessary to complete a fetal blood sample.

The tray contains a LiNA™ single handed FSBS wand, four heparinised capillary tubes, capillary caps, mixing wires and a series of universal blood gas analyser adapters. The adapters will fit the majority of commonly available blood gas analysers. The tray is wrapped in a sterile surgical drape for easy dispensing.

The capillary tube is sodium heparinised 240 µl
Inner diameter: 1.4mm and the outer diameter: 2.0mm
Length of capillary tube: 85mm
Capillary volume: minimum 130µl

Principals of Operation

The disposable LiNA™ Fetal Blood Sampler is used in the following way:

- the knife at the end of the sampler is pushed out to penetrate the scalp;
- the capillary tube is used to collect a blood sample;
- the LiNA™ Fetal Blood Sampler is supplied with three extra capillary tubes if more samples are required.

Ordering Information

Part Number	4201-AC
Quantity	5 units per case
Description	Complete Fetal Scalp Sampling Kit
Delivery	Usually ex-stock

Technical Specifications 4201-AC

The tray contains the following*	
1	LiNA™ FBS 4201 Fetal Blood Sampler
6	Long cotton tipped buds
4	Heparinised capillary tubes
8	Capillary caps
4	Mixing wires
1	Universal blood gas analyser adapter with clot buster
1	Sterile surgical drape
A resting space for ELA-01 Amnioscope	
* Products are subject to change	

LiNA is the trademark of LiNA Denmark

